



# **BIOREPEELCL3® BODY Product Specifications**

**1-844-226-8277**  
**info@medsupplysolutions.com**  
**medsupplysolutions.com**



### BioRePeelCl<sub>3</sub> protocol of use

1. Clean the zone of interest with specific detersion products (acetone, surfactants...)
2. Shake the ampoule for a few seconds until colour turn green for a temporary emulsion.
3. Using a syringe, prelate the quantity of product for the treatment without removing the butyl stopper.
4. Apply the product with nitrile gloves until it is completely absorbed.
5. Let the product performs:
  - i. FND: FACE, NECK AND DECOLLETE' 3/5 minutes
  - ii. BODY: HANDS, KNEES, ELBOWS, GLUTES, FEET 5/10 minutes
  - iii. It is possible to treat every body zone up to doctor's discreption.
6. Clean the zone with a gauze soaked with water.

#### Advised quantities:

FND	Volume mL
Face	1.5-2
Neck	1-1.5
Decolleté	2-2.5

BODY	Volume mL
Hands	2 each hand: 1 back and 1 palm
Elbows	0.5-1
Glutes	4
Knees	1
Feet	1.5 each feet
Other zones	1 each dm <sup>2</sup>

BioRePeelCl<sub>3</sub> FND is applied every 7/12 days for 4 to 6 times.

BioRePeelCl<sub>3</sub> BODY is applied every 10/15 days for 4 to 6 times.